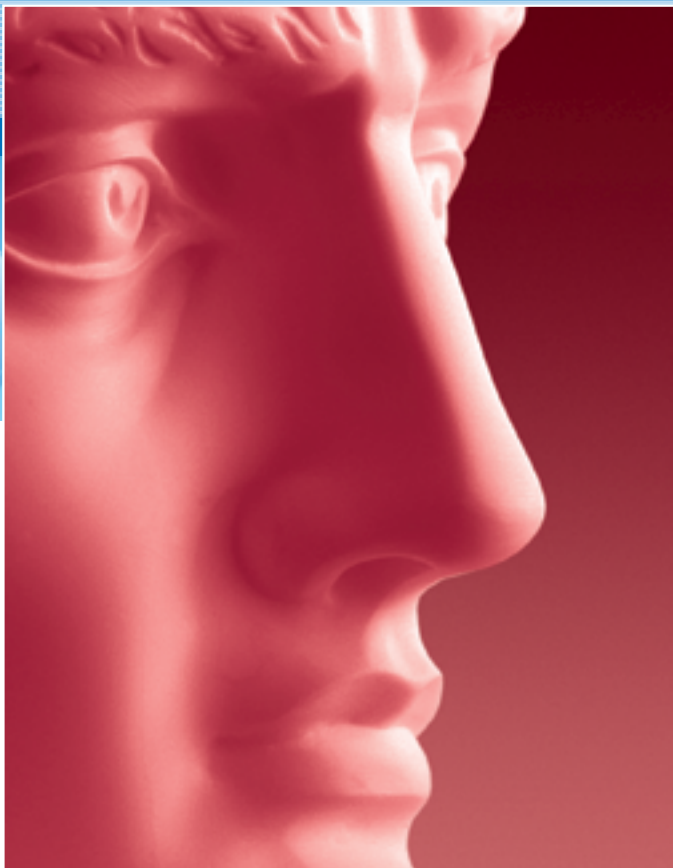
MADE IN GERMANY · 47° 57' 31.4" N · 8° 46' 58.6" E

# GUBiSCH.

Moderne Rhinoplastik  
Modern Rhinoplasty  
Rinoplastia moderna  
Rhinoplastie moderne  
Rinoplastica moderna

---

**medicon**<sup>®</sup>

---

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Nose corrections are among the most frequent operations performed in the field of cosmetic plastic surgery. However, the proportion of operations with an unsatisfactory outcome is relatively high. In addition to the experience and skill of the surgeon, another decisive factor determining the success of nose corrections is the quality of the instruments used. For a number of different reasons it would therefore appear to make sense to have available a set of instruments which is as compact and as high in quality as possible.

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In cooperation with Medicon, I have created an instrument set designed specifically with corrective nasal surgery in mind. This instrument set is continuously enlarged and adapted to new operating techniques. Particular attention was paid to ensuring that the instruments exercise the minimum possible traumatising effect but at the same time the chisels are sharp enough to easily ensure smooth cutting edges in the bone and eliminate comminuted fractures.

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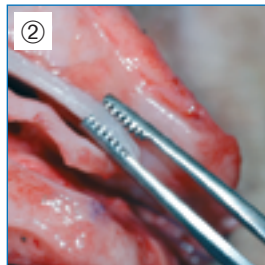
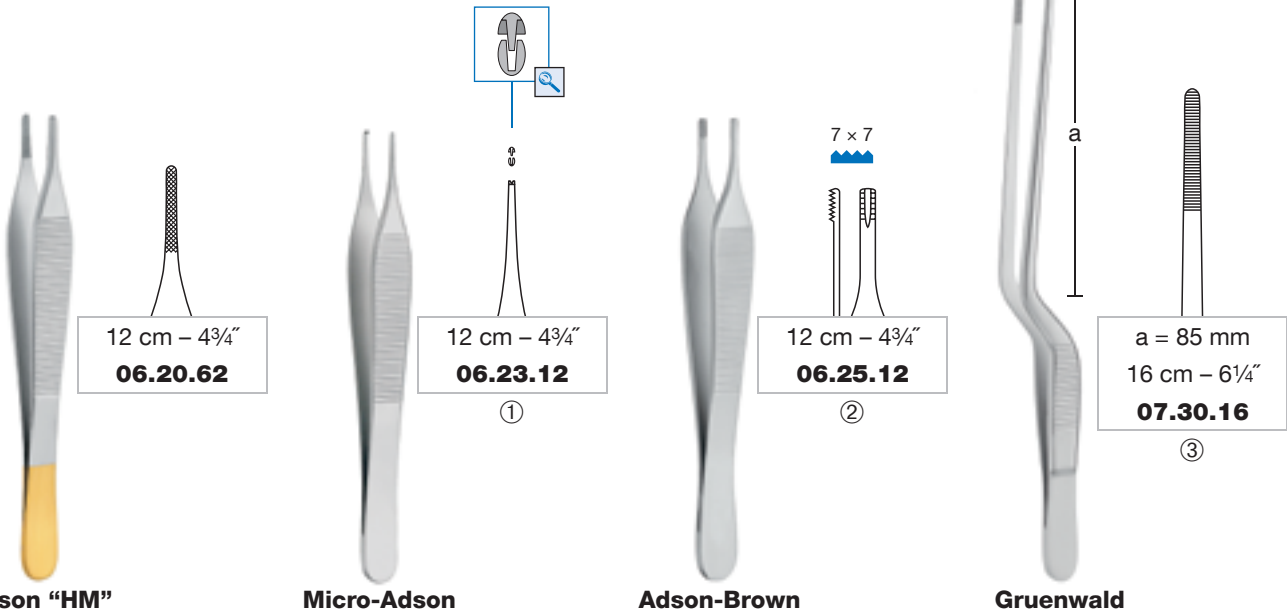
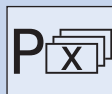
For marking incision lines on the skin and mucous membrane.

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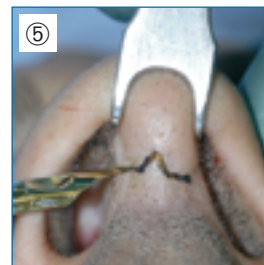
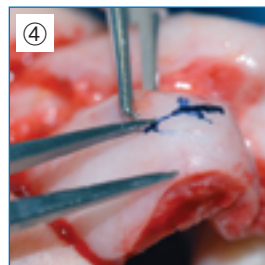
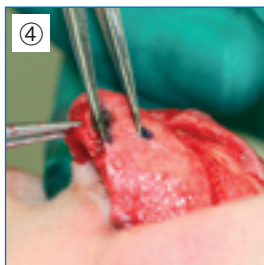
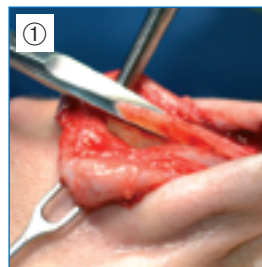
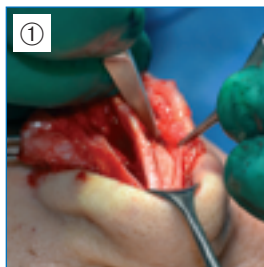
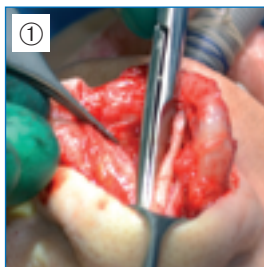
Particularly suitable for separating the lateral nasal cartilage from the septum and for using the tip to dissect the perichondrium from the lateral nasal cartilages before folding these in as "spreader flaps".

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Ideal for the preparation of soft tissue as well as cartilaginous structures.

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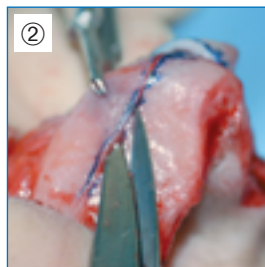
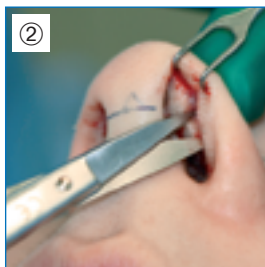
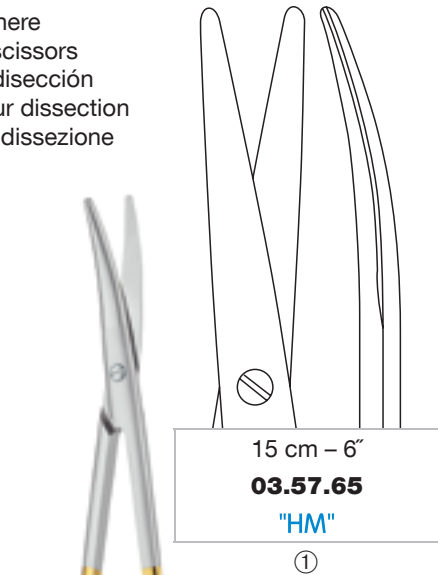
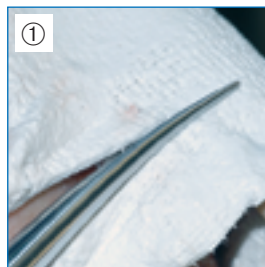
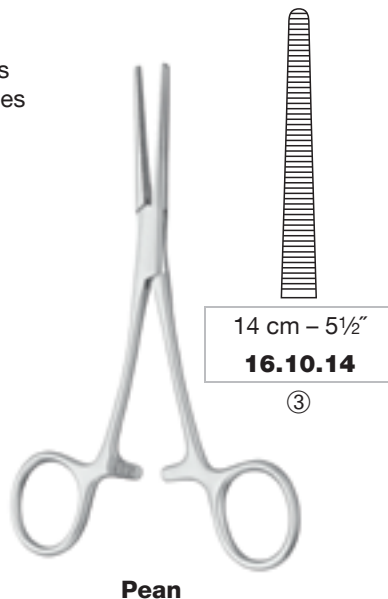
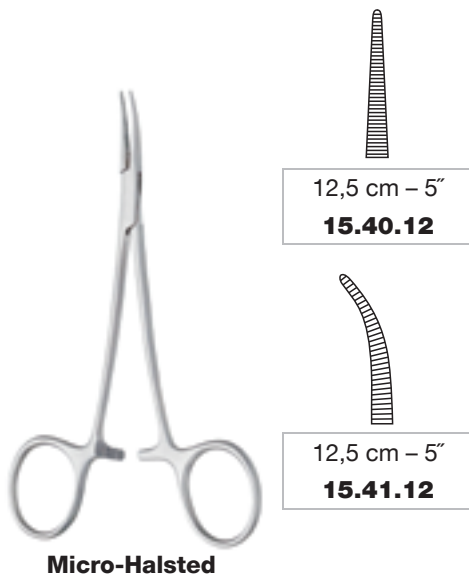
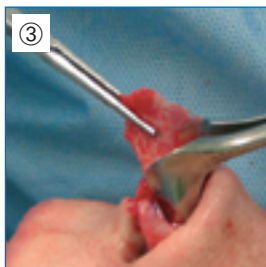
For gripping the lateral nasal cartilages to fold them under tension as spreader flaps.

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



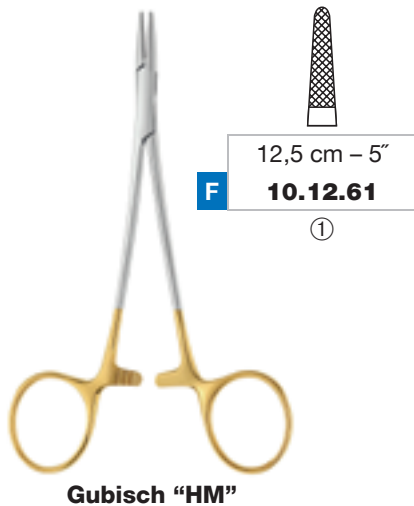
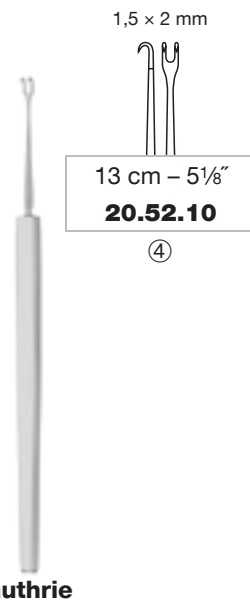
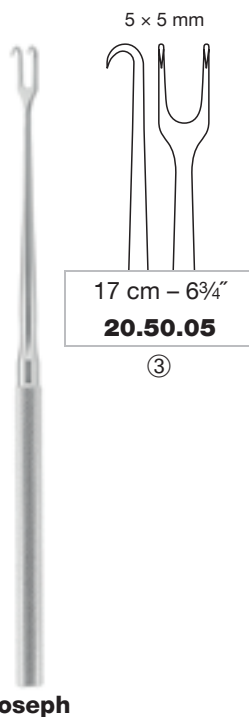
Particularly suitable for open rhinoplasty on the highly sensitive flap end.

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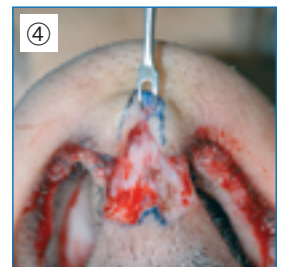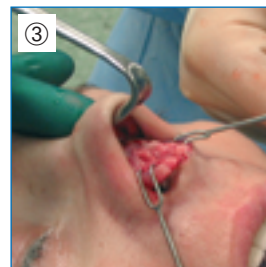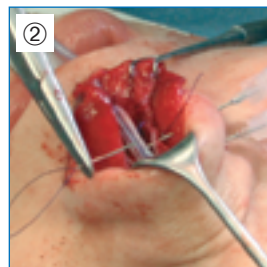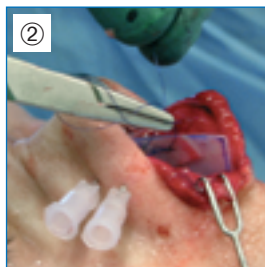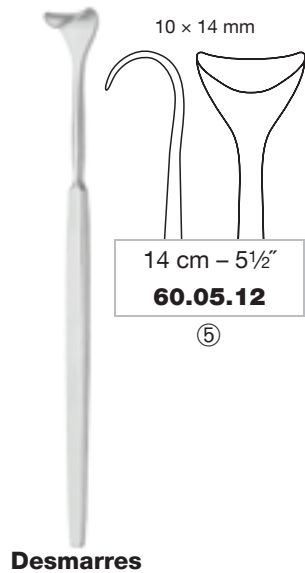
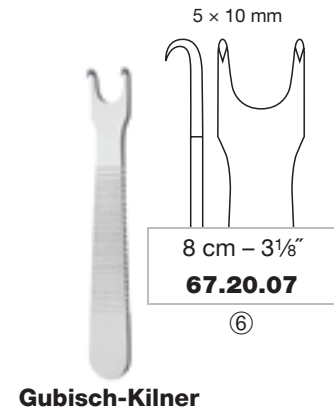
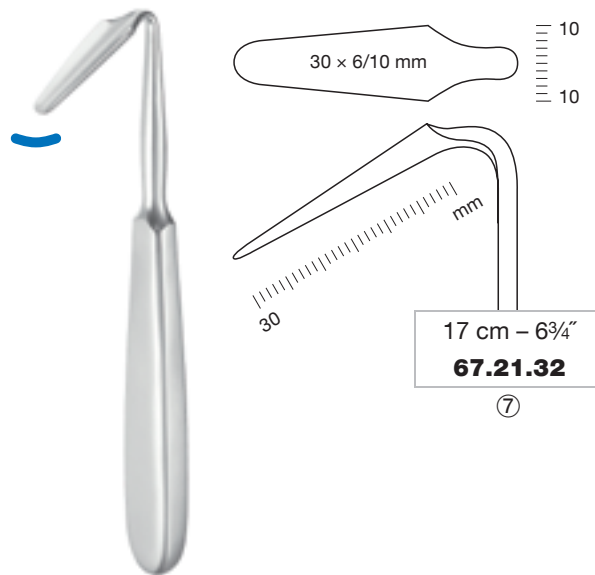
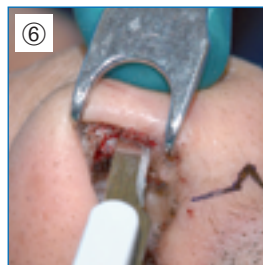
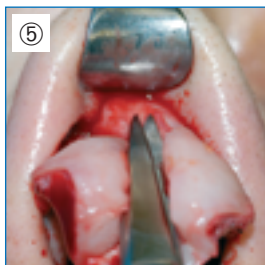
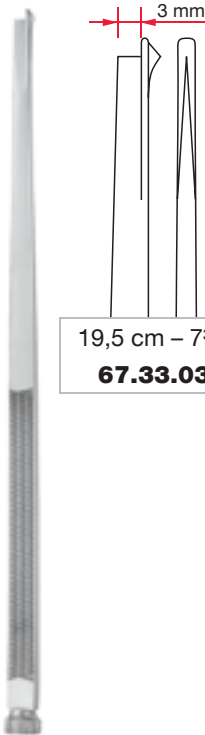
Allows dampened and precisely guided impacts without recoil.

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The surgeon orientates the position of the chisel through the lateral guiding thorn. Preferred application for lateral osteotomy with enoral and endonasal access. Minimal traumatization.

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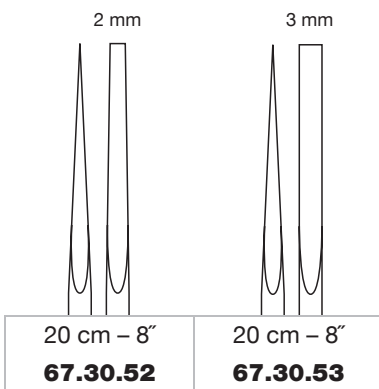
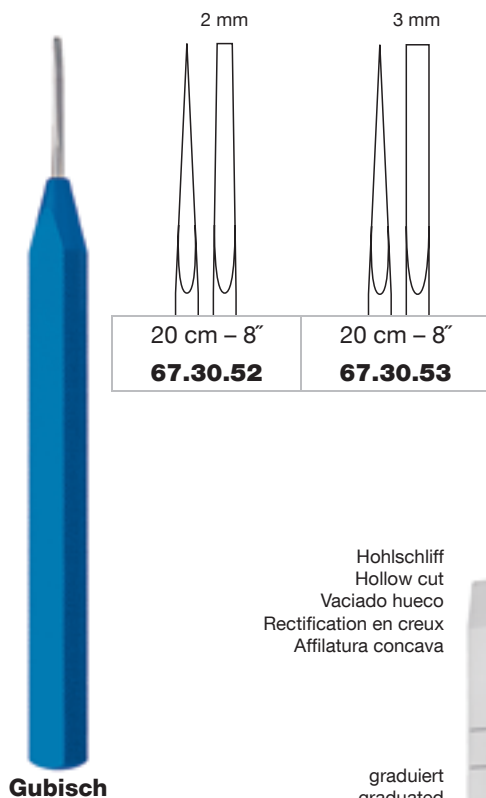
② Ideal for direct transversal and lateral osteotomies via a stab incision in the skin. Aluminium handle with wear-resistant special coating.

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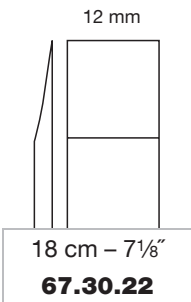
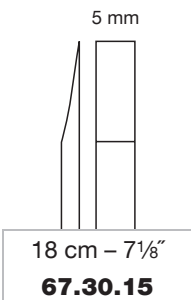
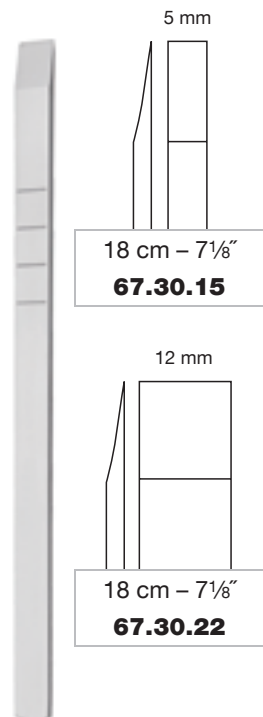
③ Extremely flat cutting edge - the set osteotomy level is continuously retained, leading to precise detachment.

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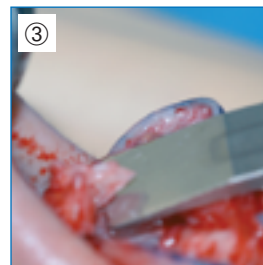
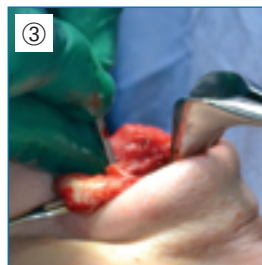
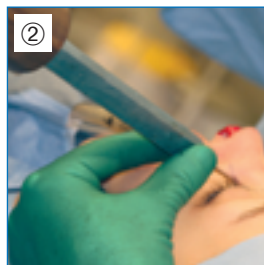
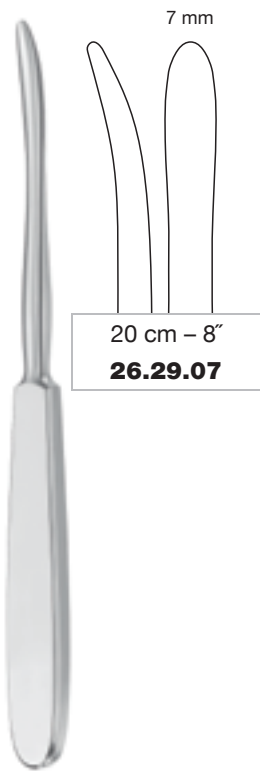
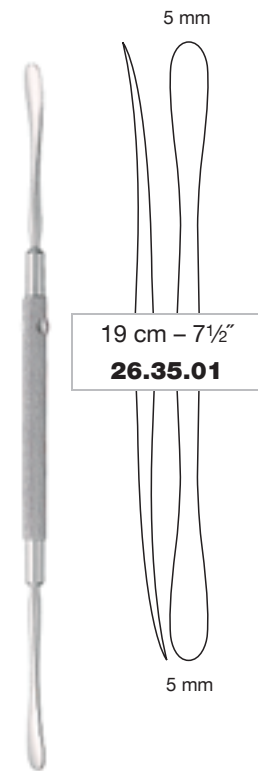
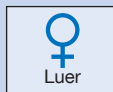
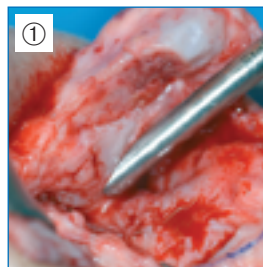
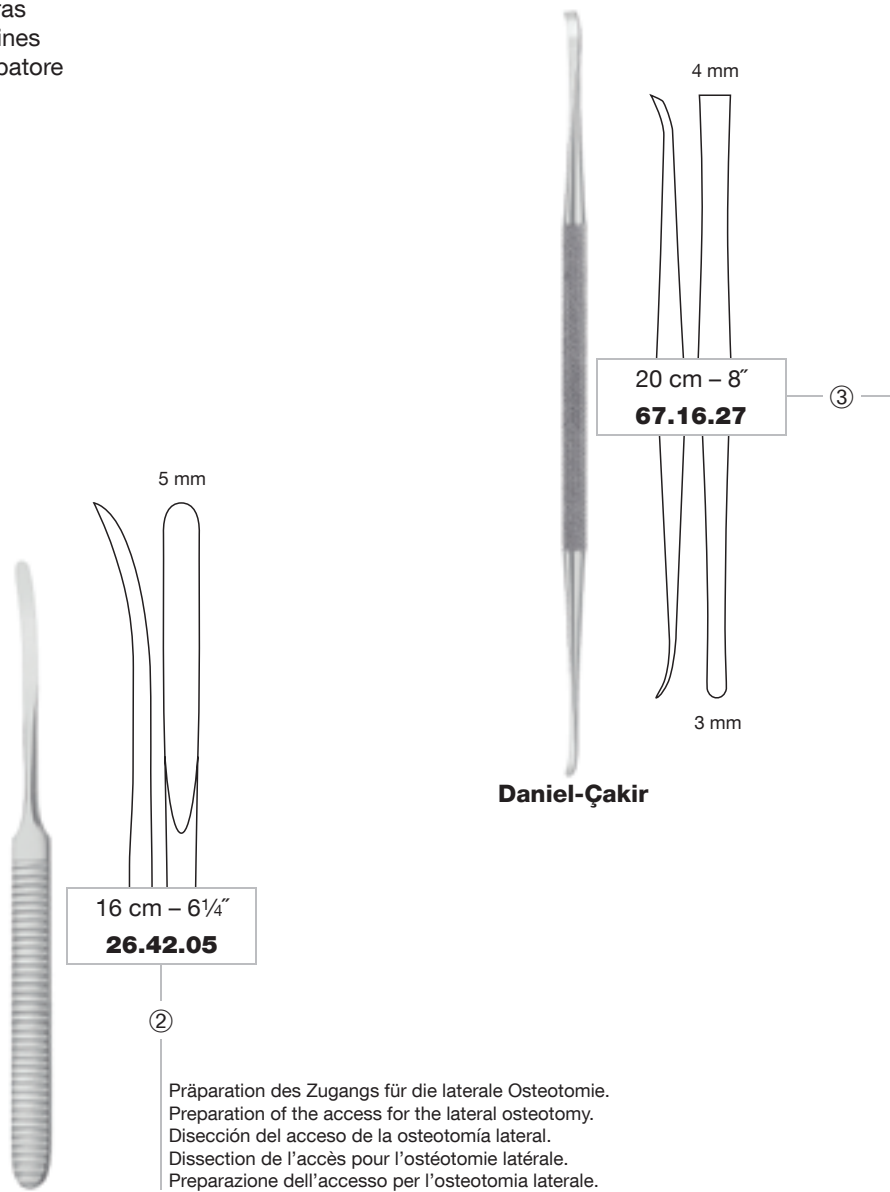
**Straight end:** Preparation of the periosteum and perichondrium in the area of the nasal dorsum.

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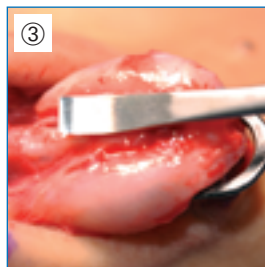
**Round end:** Preparation of the upper lateral cartilages under the bony dorsum to avoid an open roof.

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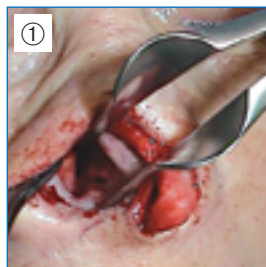
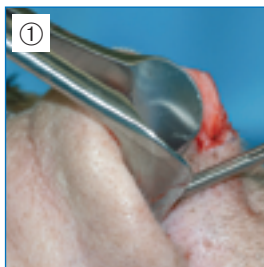
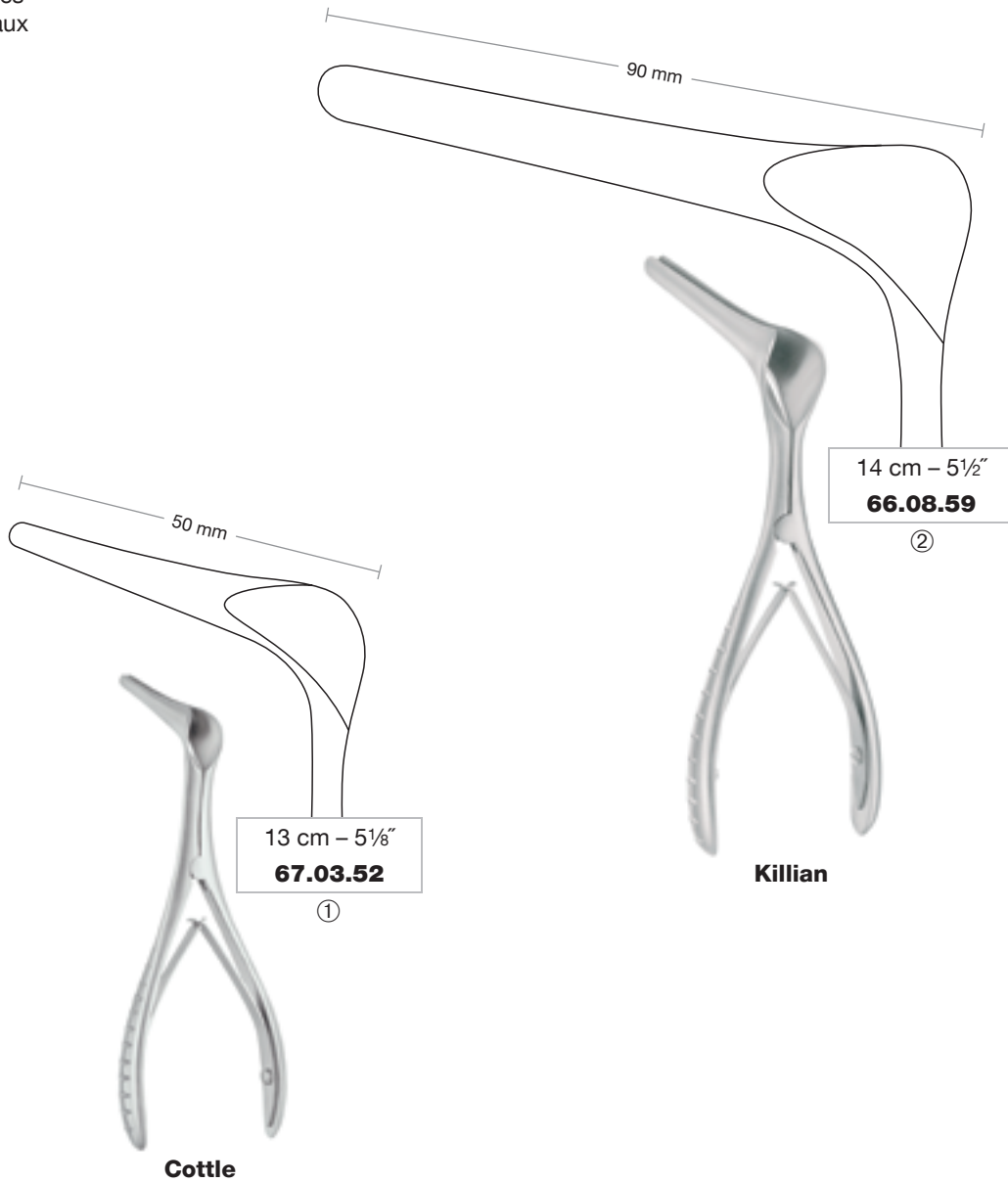
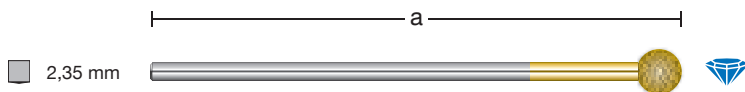
For paramedian osteotomy and to perforate the anterior nasal spine to fix the septum. To perforate the perpendicular plate with multiple holes for stabilisation of the soft cartilage.

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

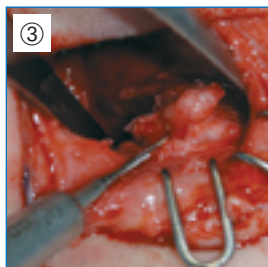
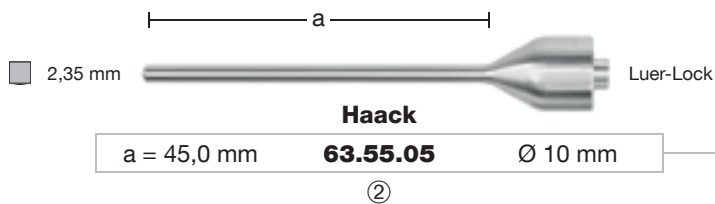
For insertion of a Luer-lock injection cannula in order to pierce the bridge of the nose and perform the newly developed technique “suture according to Haack”.

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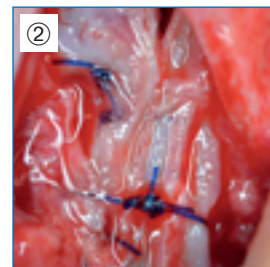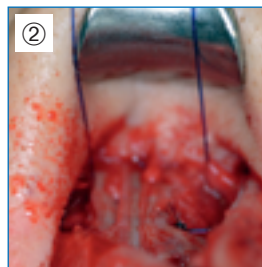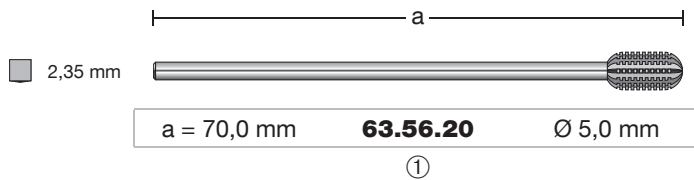
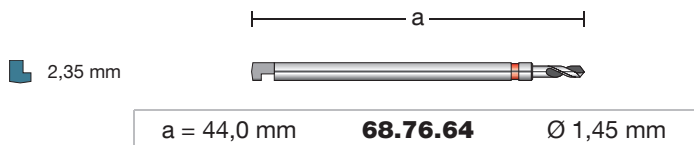
To thin out cartilage and bone, to curve cartilage by grinding on one side and to smooth the bridge of the nose after an osteotomy.

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
For perforation of the anterior nasal spine to fix the septum.

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
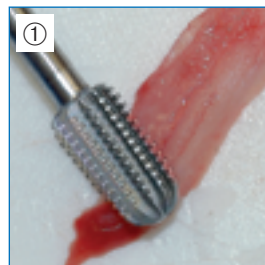
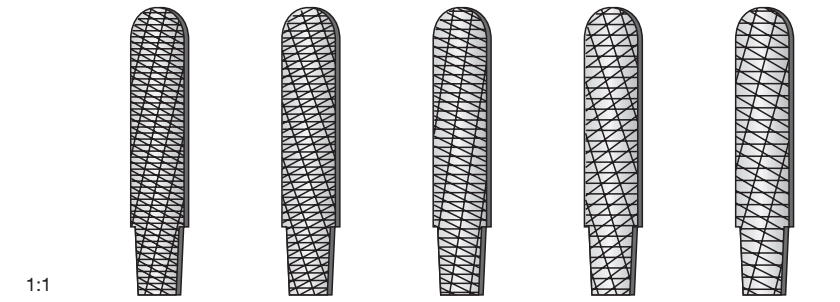
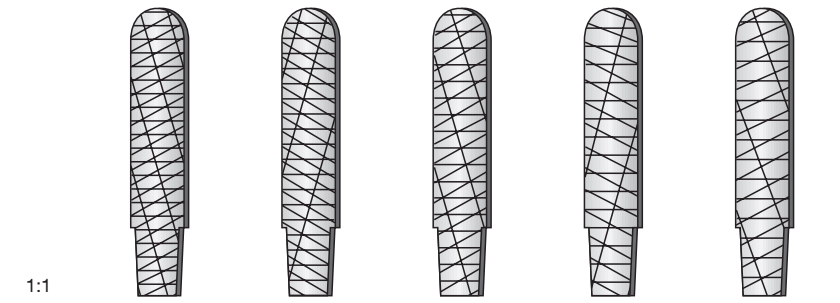
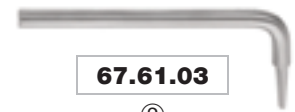
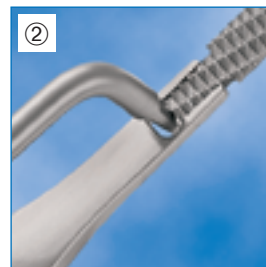
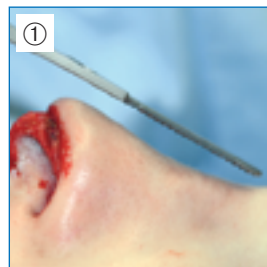
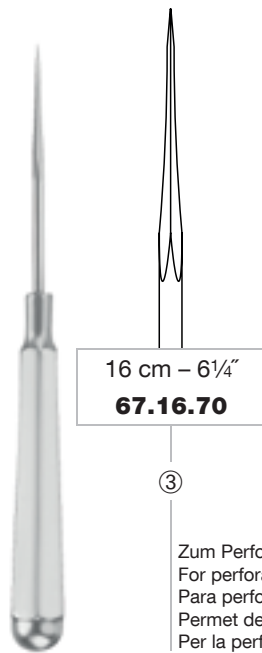
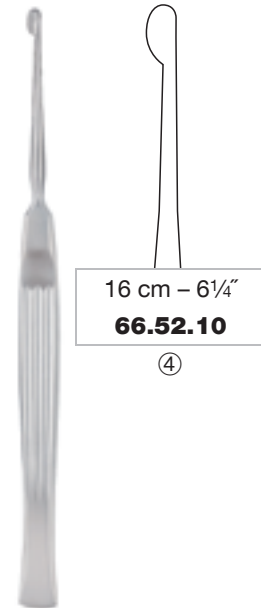
For preparation of the transition from the cartilaginous to the bony septum.

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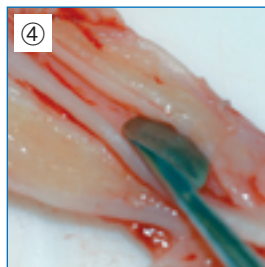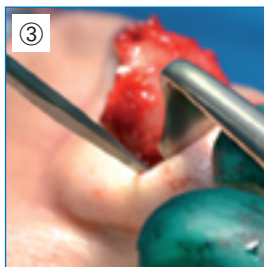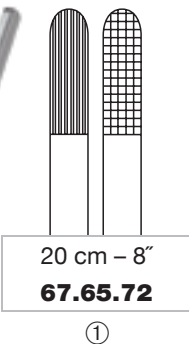
Neutralisation of tension in the cartilage permits recontouring of the cartilage without impairment to the cartilage structure.

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According to the requirements of operation, the cap may be placed either on the right or the left jaw.

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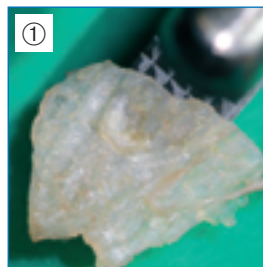
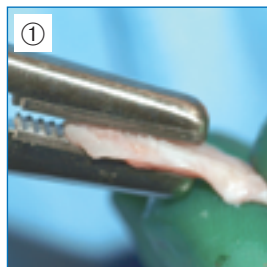
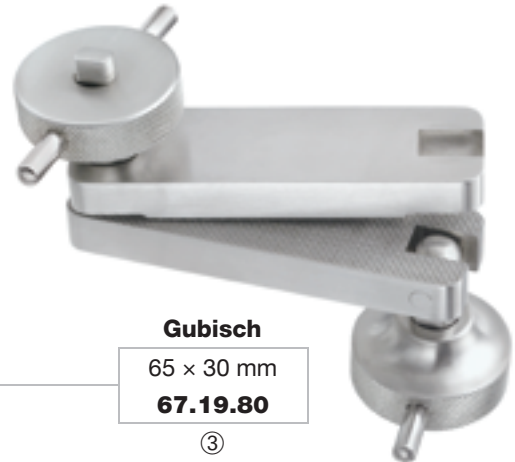
Due to an adapted pressure, a flat, flexible cartilage transplant is obtained, which can be applied in various manners.

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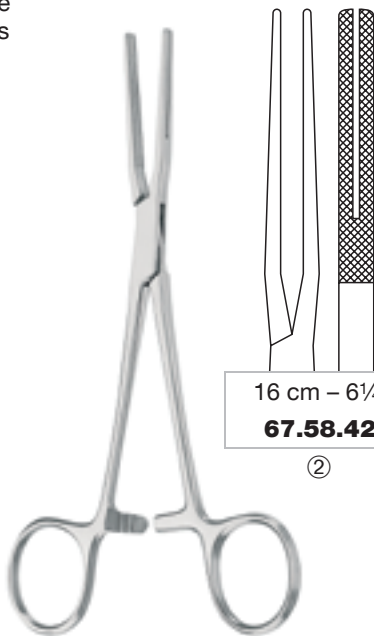
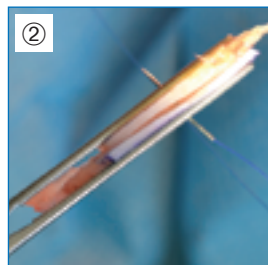
Double-layered cartilage transplants can be precisely sutured with one another. The forceps fix the transplant against the other along its entire length and prevent slipping.

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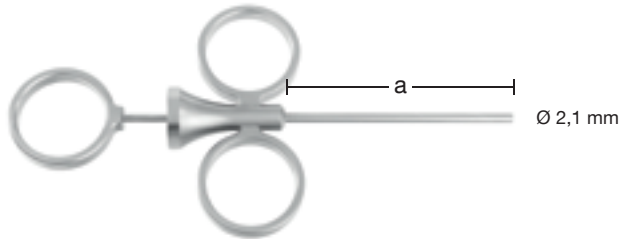
For fine modelling and discreet augmentation of the bridge of the nose using freely diced cartilage transplants.

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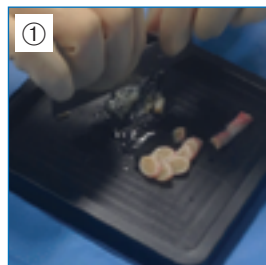
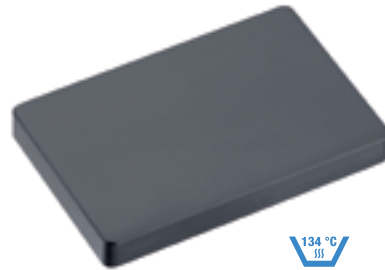
The black surface ensures particularly good contrast to the white cartilage. Ideal if the lamina perpendicularis is to be used as splint and has to be perforated multiple times for this purpose.

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For cutting of cartilage transplants in defined layer thicknesses of 0.5-2 mm.

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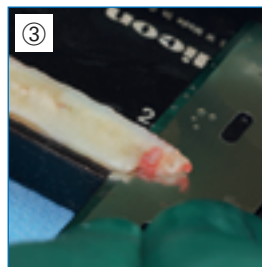
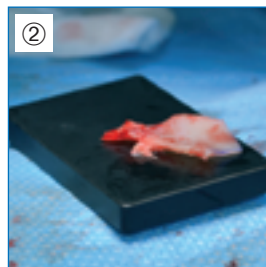
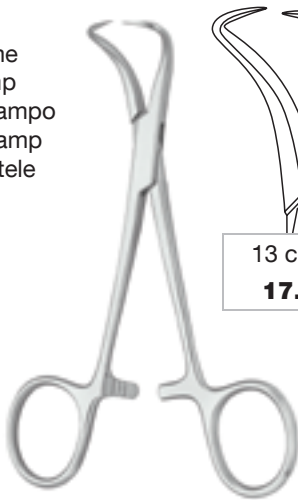
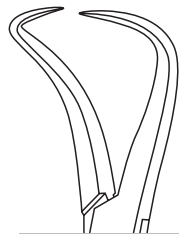
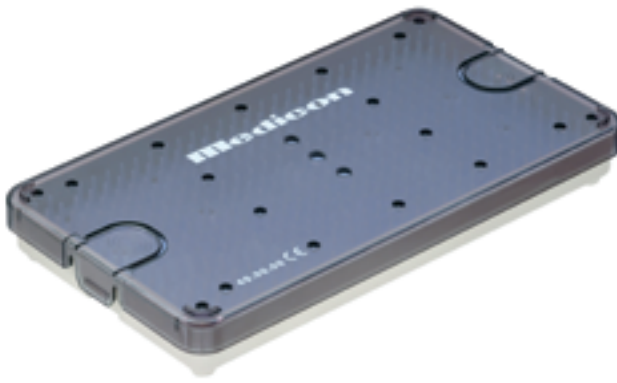
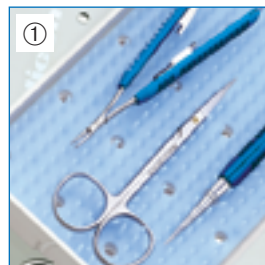
For a careful sterilization and storage of fine and delicate instruments, for example micro surgery instruments and fine hooks.

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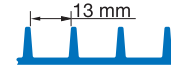
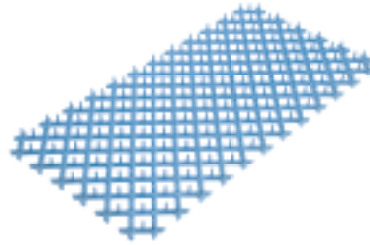
**PART 2**

| Item number | Description  | Quantity | Color |
|-------------|--|----------|-------|
| 67.16.27    | DANIEL-CAKIR Subperichondrial/subperiosteal elevator · 20 cm               | 1        |       |
| 67.15.98    | HARALDSSON Suction elevator · Ø 3 mm · 18.5 cm                             | 1        |       |
| 67.16.70    | DANIEL Spina nasalis awl · 16 cm   | 1        | ■ ■   |
| 67.19.76    | Cartilage cutting block · made of PPSU · black · 85 x 55 x 10 mm           | 1        |       |
| 67.19.79    | Cartilage cutting block with grooves · made of aluminium · 85 x 55 x 10 mm | 1        |       |
| 67.19.81    | Cartilage cutting block · made of PROPYLUX · 120 x 120 x 15 mm             | 1        |       |
| 67.19.80    | GUBISCH Cartilage crusher · 65 x 30 mm                                     | 1        | ■ ■   |
| 67.20.07    | GUBISCH-KILNER Alar retractor · 10 mm · 8 cm                               | 1        | ■ ■   |
| 67.21.32    | GUBISCH-AUFRICHT Nasal retractor · 17 cm · 30 x 6/10 mm                    | 1        | ■ ■   |
| 67.30.15    | COTTLE-GUBISCH Chisel · graduated · hollow cut · 5 mm · 18.5 cm            | 1        | ■ ■   |
| 67.30.22    | COTTLE-GUBISCH Chisel · graduated · hollow cut · 12 mm · 18.5 cm           | 1        | ■ ■   |
| 67.30.53    | GUBISCH Osteotome · with aluminium handle · 3 mm · 18.5 cm                 | 1        |       |
| → 67.30.52  | GUBISCH Osteotome · with aluminium handle · 2 mm · 18.5 cm                 | 1        |       |
| 67.33.03    | FANOUS-GUBISCH Osteotome · with thorn · 4 mm · 19.5 cm                     | 1        | ■ ■   |
| 67.53.71    | CAPLAN Nasal septum scissors · serrated cutting edge · 20 cm               | 1        | ■ ■   |
| 67.58.42    | AIACH-GUBISCH Forceps · 16 cm  | 1        | ■ ■   |
| 67.61.01    | Handle for nasal rasps · double-ended · 14 cm                              | 1        | ■ ■   |
| 67.61.03    | Wrench · for nasal rasp  | 1        |       |
| → 67.61.10  | HM Nasal rasps · figure 1  | 1        |       |
| 67.61.11    | HM Nasal rasps · figure 2  | 1        |       |
| → 67.61.12  | HM Nasal rasps · figure 3  | 1        |       |
| → 67.61.13  | HM Nasal rasps · figure 4  | 1        |       |
| 67.61.14    | HM Nasal rasps · figure 5  | 1        |       |
| → 67.61.15  | HM Nasal rasps · figure 6  | 1        |       |
| → 67.61.16  | HM Nasal rasps · figure 7  | 1        |       |
| → 67.61.17  | HM Nasal rasps · figure 8  | 1        |       |
| → 67.61.18  | HM Nasal rasps · figure 9  | 1        |       |
| → 67.61.19  | HM Nasal rasps · figure 10   | 1        |       |
| 67.65.71    | Protective guard   | 1        |       |
| 67.65.72    | GUBISCH-RUBIN HM Septal morselizer forceps · with protective guard · 20 cm | 1        | ■ ■   |
| → 68.61.05  | LINDEMANN Cutting burr · shank-Ø 2.35 mm · 44.5 mm · Ø 1.4 mm              | 1        |       |
| → 68.61.06  | LINDEMANN Cutting burr · shank-Ø 2.35 mm · 70 mm · Ø 1.4 mm                | 1        |       |
| 68.76.64    | Twist drill · STRYKER Shank end · Ø 1,45 mm · 44 mm                        | 1        |       |
| 72.00.18    | BUCHWALD Tongue spatula · 18 cm  | 1        | ■ ■   |
| 202.20.61   | SC Plus iris scissors straight · 11.5 cm                                   | 1        | ■ ■   |
| 203.51.67   | MAYO Dissecting scissors · supercut plus · curved · 17 cm                  | 1        | ■ ■   |
| 204.55.66   | KELLY Dissecting scissors · supercut plus · curved · 16 cm                 | 1        | ■ ■   |
| 266.40.21   | HEYMANN Rhinoplasty scissors · supercut · 18 cm                            | 1        | ■ ■   |

Instrumente, optional  
 Instruments optional  
 Instrumentos opcionales  
 Instruments optionnels  
 Strumenti opzionali

Das Instrument ist zusätzlich in den Farben Pink und Grün erhältlich  
 The instrument is also available in pink and green  
 El instrumento está disponible adicionalmente en los colores rosa y verde  
 L'instrument est également disponible en rose et en vert  
 Lo strumento è disponibile anche nei colori rosa e verde

**Weitere Medicon Produkte**  
**Additional Medicon Products**  
**Productos Medicon adicionales**  
**Produits Medicon supplémentaires**  
**Altri prodotti Medicon**

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The set was especially designed for plastic surgeons for obtaining rib cartilage for aesthetic and reconstructive surgery (rhinoplasty, ear reconstruction) through small incisions.

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Germany

⚠ Not all products illustrated in this documentation are available for sale in all countries.  
Please contact your local distributor for more information.

Gedruckt in Deutschland  
Printed in Germany  
Impresso en Alemania  
Stampato in R.F.G.  
Trykkes i Tyskland

Modelländerungen vorbehalten  
Patterns are subject to change  
Salvo modificaciones  
Tous droits réservés des changements de modèle  
Ci riserviamo la facoltà di cambiamenti nei modelli

451.02.06 © Copyright 04/2019, MEDICON eG, Tuttlingen, Germany

**medicon**<sup>®</sup>